

# Topigenix



## Topigenix Wound Gel (hydrogel)

5ml Tube - Box of 10 Tubes

- Hydrogel
- Covers wound
- Absorbs wound exudate
- Controls bleeding
- Protects against abrasion, friction and desiccation
- Protects against contamination
- Non-toxic, Non-sensitizing, Non-irritating

**Indications:** Topigenix Wound Gel is a hydrogel intended to manage dermal lesions such as lacerations, post-operative (surgical) wounds, partial and full-thickness wounds, 1st and 2nd degree burns and ulcers (diabetic, venous stasis, pressure). Topigenix Wound Gel can be used in conjunction with conventional dressings and/or dressings that absorb fluid (i.e. gauze, gel, alginate, foam and hydrocolloid).

**Directions:** The Topigenix Wound Gel is applied onto the affected wound area of the skin. The components of the Wound Gel allow it to be used on mucous membranes.

**For Acute Wounds:** Apply Topigenix Wound Gel directly into the wound cavity or incision. With wounds such as incisions or puncture wounds, it is recommended that the wound first be cleaned with Topigenix Skin & Wound Cleanser and that the cavity or incision then be filled with Topigenix Wound Gel. Topigenix Cream should be applied to the wound margins to provide nutrients promoting healing. The wound should then be covered with a dressing.

**For Chronic Wounds:** When treating chronic wounds, such as pressure ulcers, the open wounds and surrounding tissues should be cleansed with Topigenix Skin & Wound Cleanser and the wound bed should be completely filled with Topigenix Wound Gel. Topigenix Wound Cream should then be applied to the surrounding wound margins to provide additional nutrients to these tissues and to prevent wound expansion. The wound should then be covered with a dressing. This treatment process should be repeated every other day or as directed.

**Contraindications:** Topigenix Wound Gel is contraindicated in the following cases:

- Known or suspected allergic reactions to any of the ingredients
- Third degree burns
- Any wound where the dermis is severely damaged or is missing.

**Warning:** For external use only. Do not use in the eyes.

**Shelf Life, Packaging and Storage:** Shelf life of 36 months (3 years). Package of 10, single patient use, 5 ml tubes. Store at controlled room temperature (41-86° F).

Manufactured in the USA exclusively for:  
Healthcare Industry Group, Inc., Naples, FL  
[www.healthcareindustrygroup.com](http://www.healthcareindustrygroup.com)  
[Topigenix@healthcareindustrygroup.com](mailto:Topigenix@healthcareindustrygroup.com)  
Phone: 630-450-0639

**HIG** Healthcare Industry Group